# MEDICAL PROGRAMM II

2019/2020

Sicher. Sauber. **ALPRO.**

## ALPRO MEDICAL GMBH

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**ALPRO** is one of the most creative companies in the infection control sector. **ALPRO** stands out due to first class and environment-friendly products and reliability. Furthermore,

**ALPRO** is one of the world's leading specialists in the cleaning, maintenance and disinfection of dental suction systems with and without amalgam separator (AlproJet-D, AlproJet-DD, AlproJet-W, AlproCleaner).

Aldehyde-free products are a matter of course for us; including the **ALPRO**-products, practice-proven for years, MinutenSpray-classic and MinutenWipes variants for the quick disinfection of surfaces, the universally applicable products BIB forte eco (burr and instrument bath) for the disinfection and the cleaning of all instruments as well as PlastiSept eco/Wipes variants, the current gentle alcohol-, aldehyde- and QAC-free disinfectant for all plastic and metal surfaces.

**ALPRO** is a global leader in the research and the development of products for the removing of biofilm in dental unit water lines and the decontamination of procedural water. With BRS® (Biofilm-Removing-System) and Alpron/Bilpron the requirements of EN ISO 16954:2015 are met and drinking water quality in dental units is achieved. In order to maintain this standard, which is necessary for the dental practice, two different systems for the decontamination of procedural water, Weekend-System II and ALPRO-BCS (BottleCareSystem), are available for the connection to the dental unit.

A real highlight is the WL-line, cleaning and disinfection solutions for manual processing of handpieces and turbines.

These solutions comply with the highest cleaning (EN ISO 15883) and disinfection requirements and were tested and approved by several manufacturers.

## ALPRO MEDICAL GMBH

Registered as manufacturer of system participating obliged packaging

The partner for the disposal of our sales packaging

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- PZN (für 300 ml): 11857502
- PZN (für 1 L): 3645335

### Active ingredient basis

Modern combination of biguanide and tosylchloramide

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*Rinsing solution ready for use for application in mouth rinsing  
and ultrasonic devices.*

*Alprox is used for the cleaning and care of the oral cavity as well  
as gingival pockets during and after the treatment of paradontosis.  
Alprox can also be used as rinsing solution for the cleaning and  
care of gingival pockets during/after removal of tartar and concre-  
ments and for rinsing the root canal.*

*Alprox has no negative effect on the action of detergents and  
disinfectants in the aspiration system.*

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1 L\*

#### Efficacy/Application

- **bactericidal** incl. MRSA and suitable for hospitalism prophylaxis
- **yeastocidal** (*C. albicans*)
- **virus inactivating** limited spectrum of virucidal activity: **all enveloped "blood-borne" viruses** as well as non-enveloped adeno-, rota-, noro-viruses
- proved according to EN 1500, EN 14476
- hygienic hand disinfection 30 sec.

#### Active ingredient basis

Ethanol, propan-2-ol, skin care substances, QAC + Alkylamines; HDE gel incl. thickener.

## Cleaning, disinfection and care of hands

### AlproSept-HDE/AlproSept-HDE gel

Hand disinfection as well as skin disinfection before carrying out ordinary injections and punctures of peripheral vessels. AlproSept-HDE and AlproSept-HDE gel have a particularly good cutaneous tolerance. Free of aldehydes and phenols. AlproSept-HDE/HDE gel is applied undiluted.

- For hygienic hand disinfection, rub 3 - 5 ml AlproSept-HDE/HDE gel and keep moist for 30 seconds.

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### Application notes

#### Standard rubbing method for hygienic hand disinfection according to EN 1500

For the **hygienic hand disinfection** take the hand disinfectant into your cupped, dry hands and rub for **30 seconds** up to the wrists following the listed steps. The moves of each step have to be repeated five times. After finishing step 6 repeat the single steps until the indicated rubbing time has been attained.

**Pay attention to keep the hands moist over the entire rubbing time.**

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#### Step 1 approx. 5 seconds

Rub the hand palms together, additionally the wrists

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#### Step 2 approx. 5 seconds

Rub right hand palm against the back of the left hand and vice versa

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#### Step 4 approx. 5 seconds

Rub the outsides of the folded fingers against the opposite palm

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#### Step 5 approx. 5 seconds

Rub the left thumb in circles in the closed right hand and vice versa

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#### Step 6 approx. 5 seconds

Move the closed fingertips in circles in the right palm and vice versa

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### Application notes

#### Standard rubbing method for surgical hand disinfection according to EN 12791

For the **surgical hand disinfection** take the hand disinfectant into your cupped, dry hands and rub for **3 minutes** up to the elbows following the listed steps.

**Pay attention to keep the hands moist over the entire rubbing time.**

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#### Step 1

Apply hand disinfectant on dry hands. Use elbow to operate the dispenser.

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#### Step 2

Rub forearms up to the elbows with hand disinfectant for **at least 3 minutes**.

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#### Step 3

Disinfect hands. Follow rubbing method for hygienic hand disinfection.

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#### Step 4

Keep hands above elbow level during and after disinfection. Hands must be air-dried before wearing surgical gloves.

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*Mild washing lotion for frequent hand washing. Cleans the skin particularly gently. Contains skin protection substances that maintain the natural acid mantle of the skin. CREMANA®-wash is pH-neutral and has a reserved scent.*

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#### Field of application

Ideal for the use in all medical fields that require frequent hand washing, e. g. medical and dental practices, hospitals, etc.

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#### Application and dosage

After the cleaning/decontamination of the hands (breaks/end of work) put a cherry stone sized on the back of the hand and rub in thoroughly.

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*Cream for intensive skin protection and care. CREMANA® Creme-Lotion is easy to rub in, absorbs quickly and does not grease. Suitable for all skin types.*

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*Greasing skin protection and barrier cream (W/O) free of silicone, for application at work in case of frequent contact with water based products, like cleaning and disinfection agents. Ideal for skin protection when wearing gloves.*

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### Application and dosage


After the cleaning/decontamination of the hands (breaks/end of work) put the barrier creme (cherry stone sized) on the back of the hand and rub thoroughly.

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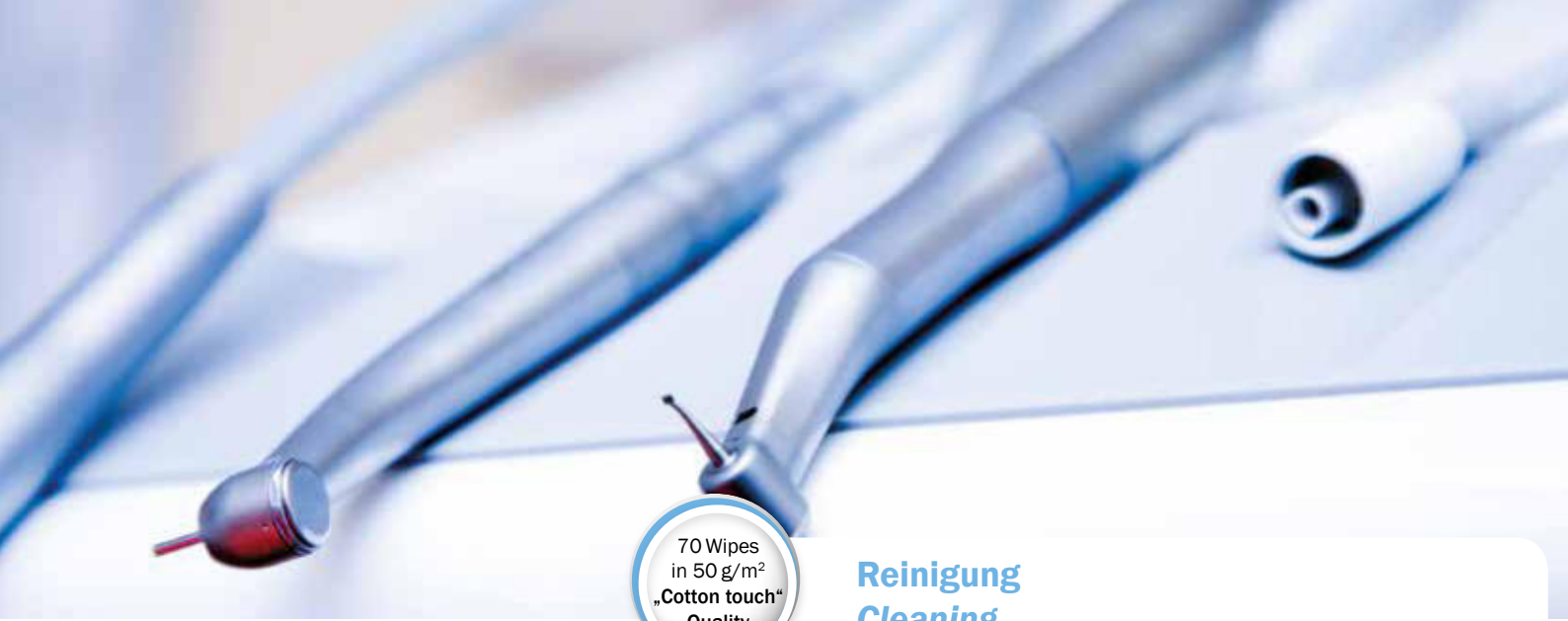#### Efficacy/Certification

- cleaning/decontaminating
- bacteriostatic
- fungistatic
- dermatologically approved

#### Active ingredient basis

EDTA, biguanides in glycolic-aqueous solution.

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Alcohol-free wet wipes for the non protein fixing cleaning and decontamination of surfaces of medical devices and intact skin.  
The bacteriostatic and fungistatic acting CleanWipes in „cotton touch“ quality, remove easily and carefully organic and inorganic soiling from medical devices, hands and intact skin.  
CleanWipes are dermatologically tested.

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*Alcoholic solution ready for use for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment.  
Approved amongst others by Dentsply Sirona, Planmeca.  
Only for use on alcohol compatible surfaces.*

### Contact times (undiluted)

- bactericidal (incl. TBC, MRSA) 1 min.
- yeasticidal 1 min.
- virus inactivating limited spectrum of virucidal activity according to EN 16777:
  - » all enveloped „blood-borne“ viruses 30 sec.
  - » as well as non-enveloped viruses (see efficacy) 2 min.

### Active ingredient basis

Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.

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### Efficacy/Certification

- **bactericidal** incl. TBC, MRSA and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777: non-enveloped adeno-, noroviruses as well as all enveloped „blood-borne“ viruses, such as HBV, HCV, HIV and influenza
- with remanence effect (inhibition of recontamination)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection

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#### Efficacy/Certification

- **bactericidal** incl. TBC, MRSA and suitable for hospitalism prophylaxis
- **yeastcidal**
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777: non-enveloped adeno- and noroviruses as well as all enveloped „blood-borne“ viruses, such as HBV, HCV, HIV and influenza
- active solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection in the medical area

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- REF 3044** **MinutenWipes 20**,  
Spenderbox mit 120 Tücher\*/dispenser box with 120 wipes\*
- REF 3045** **MinutenWipes 20**,  
Karton mit 6 NF-Beutel\*/carton with 6 refill-bags\*
- REF 3046** **MinutenWipes Jumbo 20**,  
Spenderbox mit 200 Tücher\*/dispenser box with 200 wipes\*
- REF 3040** **MinutenWipes Jumbo 20**,  
Karton mit 12 NF-Beutel\*/carton with 12 refill-bags\*
- REF 4602** Wandhalter für Jumbo-Wipes-Box\*  
Wall dispenser for Jumbo-Wipes-Box\*

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*Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment. Approved amongst others by Dentsply Sirona, Planmeca.*

#### Contact times (undiluted)

- **bactericidal** (incl. TBC, MRSA) 1 min.
- **yeastcidal** 1 min.
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777:
  - » all enveloped „blood-borne“ viruses 30 sec.
  - » as well as non-enveloped viruses (see efficacy) 2 min.

#### Active ingredient basis

Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.

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*Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment. Approved amongst others by Dentsply Sirona, Planmeca.*

### Contact times (undiluted)

- bactericidal (incl. TBC, MRSA) 1 min.
- yeasticidal 1 min.
- virus inactivating limited spectrum of virucidal activity according to EN 16777:
  - » all enveloped „blood-borne“ viruses 30 sec.
  - » as well as non-enveloped viruses (see efficacy) 2 min.

### Active ingredient basis

*Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.*

Oberfläche/Surface of  
MinutenWipes 50  
(à 50 g/m<sup>2</sup>, 14 x 20 cm)  
MinutenWipes Jumbo 50  
(à 50 g/m<sup>2</sup>, 21 x 26 cm)

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### Efficacy/Certification

- **bactericidal** incl. TBC, MRSA and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777: non-enveloped adeno- and noroviruses as well as all enveloped „blood-borne“ viruses, such as HBV, HCV, HIV and influenza
- active solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection in the medical area

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- REF 4709** MinutenWipes 50, StarterSet  
2 Spenderboxen, leer mit 2 NF-Beutel\*/  
2 dispenser boxes, empty with 2 refill-bags\*
- REF 4710** MinutenWipes 50,  
Karton mit 6 NF-Beutel\*/carton with 6 refill-bags\*
- REF 4711** MinutenWipes Jumbo 50,  
Spenderbox mit 70 Tücher\*/dispenser box with 70 wipes\*
- REF 4712** MinutenWipes Jumbo 50,  
Karton mit 12 NF-Beutel\*/carton with 12 refill-bags\*
- REF 4602** Wandhalter für Jumbo-Wipes-Box\*  
Wall dispenser for Jumbo-Wipes-Box\*

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**Efficacy/Certification**

- **bactericidal** incl. TBC, MRSA and suitable for hospitalism prophylaxis
- **yeastical**
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777: non-enveloped adeno- und noroviruses as well as all enveloped „blood-borne“ viruses such as HBV, HCV, HIV and influenza
- active solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection in the medical area

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*Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment. Approved amongst others by Dentsply Sirona, Planmeca.*

**Contact times (undiluted)**

- **bactericidal** (incl. TBC, MRSA) 1 min.
- **yeastical** 1 min.
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777:
  - » all enveloped „blood-borne“ viruses 30 sec.
  - » as well as non-enveloped viruses (see efficacy) 2 min.

**Active ingredient basis**

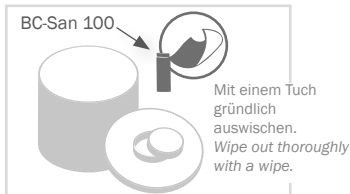
Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.

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
### Step 1 - Disinfection

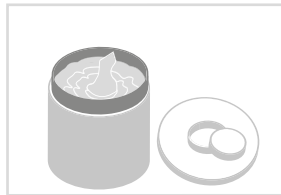Dispose of solution and wipes beyond expiry date. Wipe thoroughly the inside of the empty box and the lid for processing with a cloth soaked in BC-San 100.

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
### Step 2 - Filling

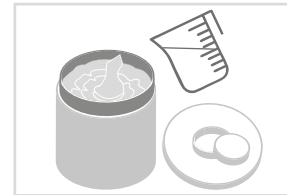Put the unpacked reel with bag into the box. Pull out a small piece of the innermost wipe from the middle.

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### Step 3 - Soaking

Then slowly pour the ready for use solution (MinutenSpray-classic or PlastiSept eco) spirally from the middle outwards over the reel. Thus you ensure a complete wetting.

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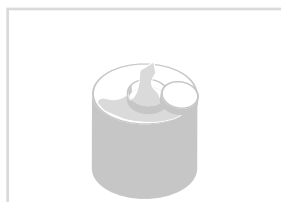
### Step 4 - Stick through

Now pull the innermost wipe of the reel through the star-shaped tear-off slot of the lid and throw it away.

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
### Step 5 - Closing

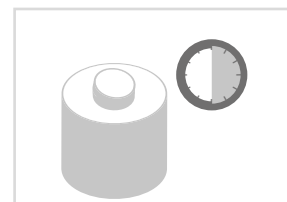Put the lid on the dispenser and close until it locks.

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### Step 6 - Soaking time

For complete soaking it is sufficient to let the wipes rest for 30 minutes.

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### Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeastcidal**
- **virus inactivating** according to EN 16777, active against all enveloped incl. „blood-borne“ viruses, such as HBV, HCV, HIV, influenza, BVDV, vaccinia as well as non-enveloped noro-viruses (120 min.)
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), surface disinfection with mechanics

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- PlastiSept eco**
- REF 4645** 500 ml\*
- REF 4646** 5 L Canister\*
- REF 4647** 5 L Cubitainer\*

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*Alcohol-free solution ready for use for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines, particularly with acrylic glass or metal surfaces.*

### Contact times (undiluted)

- |   |        |
|---|--------|
| • bactericidal  | 1 min. |
| • yeastcidal  | 1 min. |
| • TBC   | 5 min. |
| • virus inactivating according to EN 16777 active against:<br>» all enveloped incl. „blood-borne“ viruses | 5 min. |

### Active ingredient basis

Current, alcohol-free, aldehyde-free and QAC-free formulation based on sec. and tert. alkylamines.

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*Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines, particularly with acrylic glass or metal surfaces.*

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### Active ingredient basis

Current, alcohol-free, aldehyde-free and QAC-free formulation based on sec. and tert. alkylamines.

Oberfläche/Surface of  
PlastiSept eco Wipes 20  
(à 20 g/m<sup>2</sup>, 14 x 20

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### Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virus inactivating** according to EN 16777, active against all enveloped incl. „blood-borne“ viruses, such as HBV, HCV, HIV, influenza, BVDV, vaccinia as well as non-enveloped noro-viruses (120 min.)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), surface disinfection with mechanics

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**REF 4636 PlastiSept eco Wipes 20**, Spenderbox mit 120 Tücher\*/dispenser box with 120 wipes\*

**REF 4637 PlastiSept eco Wipes 20**, Karton mit 20 NF-Beutel\*/carton with 20 refill-bags\*

**REF 4638 PlastiSept eco JumboWipes 20**, Spenderbox mit 200 Tücher\*/dispenser box with 200 wipes\*

**REF 4639 PlastiSept eco JumboWipes 20**, Karton mit 12 NF-Beutel\*/carton with 12 refill-bags\*

**REF 4602** Wandhalter für Jumbo-Wipes-Box\*  
Wall dispenser for Jumbo-Wipes-Box\*

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*Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines, particularly with acrylic glass or metal surfaces.*

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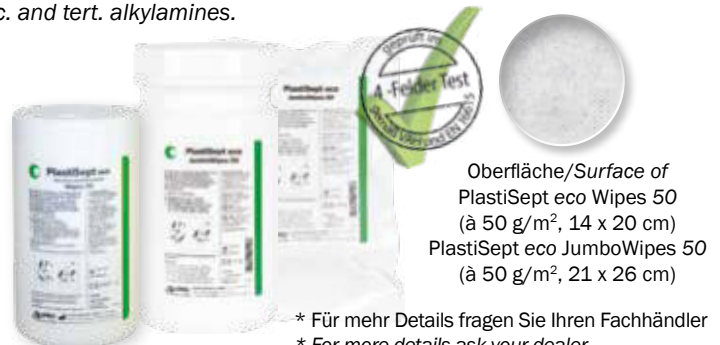
### Active ingredient basis

*Current, alcohol-free, aldehyde-free and QAC-free formulation based on sec. and tert. alkylamines.*

Oberfläche/Surface of  
PlastiSept eco Wipes 50  
(à 50 g/m<sup>2</sup>, 14 x 20 cm)  
PlastiSept eco JumboWipes 50  
(à 50 g/m<sup>2</sup>, 21 x 26 cm)

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### Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virus inactivating** according to EN 16777, active against all enveloped „blood-borne“ viruses, such as HBV, HCV, HIV, influenza, BVDV, vaccinia as well as non-enveloped noro-viruses (120 min.)
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), surface disinfection with mechanics

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- REF 4642** PlastiSept eco Wipes 50, StarterSet  
2 Spenderboxen, leer mit 2 NF-Beutel\*/  
2 dispenser boxes, empty with 2 refill-bags\*
- REF 4643** PlastiSept eco Wipes 50,  
Karton mit 6 NF-Beutel\*/carton with 6 refill-bags\*
- REF 4623** PlastiSept eco JumboWipes 50,  
Spenderbox mit 70 Tücher\*/dispenser box with 70 wipes\*
- REF 4624** PlastiSept eco JumboWipes 50,  
Karton mit 12 NF-Beutel\*/carton with 12 refill-bags\*
- REF 4602** Wandhalter für Jumbo-Wipes-Box\*  
Wall dispenser for Jumbo-Wipes-Box\*

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*Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines, particularly with acrylic glass or metal surfaces.*

Approved amongst others by Dentsply Sirona.

### Contact times (undiluted)

- |   |        |
|---|--------|
| • bactericidal  | 1 min. |
| • yeasticidal   | 1 min. |
| • TBC   | 5 min. |
| • virus inactivating according to EN 16777 active against:<br>» all enveloped incl. „blood-borne“ viruses | 5 min. |

### Active ingredient basis

Current, alcohol-free, aldehyde-free and QAC-free formulation based on sec. and tert. alkylamines.

Oberfläche/Surface of  
PlastiSept eco MaxiWipes  
(à 50 g/m<sup>2</sup>, 17 x 26 cm)

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### Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virus inactivating** according to EN 16777, active against all enveloped incl. „blood-borne“ viruses, such as HBV, HCV, HIV, influenza, BVDV, vaccinia as well as non-enveloped noro-viruses (120 min.)
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), surface disinfection with mechanics

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#### Field of application, soaked with PlastiSept eco

Especially suited for surfaces which are sensitive to alcohol such as treatment units (e. g. upholstery, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), and surfaces of acrylic glass.

#### Field of application, soaked with MinutenSpray-classic

Suited for surfaces which are compatible with alcohol if quick drying and a comprehensive disinfection are required.

#### Active ingredient basis/Efficacy

- soaked with PlastiSept eco s. p. 20
- soaked with MinutenSpray-classic s. p. 15

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• Please make sure that the dry disposable wipes to be soaked are compatible with the used disinfection solution!

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**REF 4628** MaxiWipes reel/MaxiWipes-L reel, Spenderbox, leer\*/dispenser box, empty\*

**REF 4613** MaxiWipes reel, Karton mit 4 NF-Beutel à 250 Tücher, trocken\*/carton with 4 refill-bags of 250 wipes\*

**REF 4615** MaxiWipes-L reel, Karton mit 4 NF-Beutel à 160 Tücher, trocken\*/carton with 4 refill-bags of 160 wipes, dry\*

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Large dry wipes „cotton touch“ quality in the MaxiWipes reel-Box for the cleaning and disinfection of medical devices, especially for larger surfaces, in combination with MinutenSpray-classic or PlastiSept eco.

#### Contact times - soaked with PlastiSept eco

- bactericidal 1 min.
- yeasticidal 1 min.
- virus inactivating active against all enveloped viruses according to EN 16777:
  - » enveloped viruses 5 min.
- TBC 5 min.

#### Contact times - soaked with MinutenSpray-classic

- bactericidal (incl. TBC, MRSA) 1 min.
- yeasticidal 1 min.
- virus inactivating limited spectrum of virucidal activity according to EN 16777:
  - » enveloped viruses 30 sec.
  - » non-enveloped viruses 2 min.

Oberfläche/Surface of  
MaxiWipes reel (à 50 g/m<sup>2</sup>, 17 x 26 cm)/  
MaxiWipes-L reel (à 50 g/m<sup>2</sup>, 20 x 40 cm)

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Soak a suitable cellulose tissue with undiluted BC-San 100, wipe out all inner and outer surfaces of the dispenser box (including lid) and let it take effect for at least 5 minutes.

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
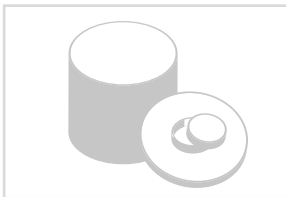
### Step 1

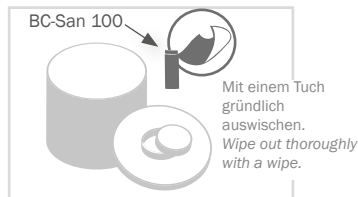Dispose of solution and wipes beyond expiry date.

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
### Step 2

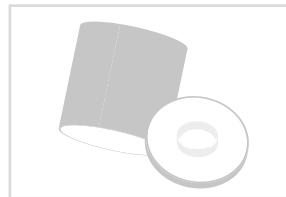Wipe thoroughly the inside of the empty box and the lid for processing with a cloth soaked in BC-San 100.

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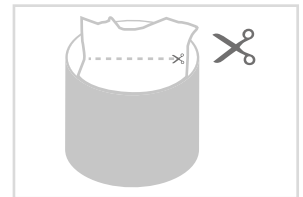
### Step 4

Place the bag into the dispenser box and cut it open along the dividing line.

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*Liquid concentrate free of aldehydes and phenols with comprehensive microbicidal efficacy for the cleaning and disinfection of surfaces of medical devices.*

### Contact times

#### • General hygiene and prophylaxis

bactericidal, fungicidal	1,0 %	60 min.
and virucidal against enveloped viruses (s. efficacy)	2.0 %	15 min.

#### • Critical areas (e. g. operating room)

bactericidal, fungicidal and virucidal (against enveloped and non-enveloped „blood-borne“ viruses)	4.0 %	60 min.
(see efficacy)	5.0 %	15 min.

### Active ingredient basis

Contemporary combination based on QAC, tensides and complexing agents.

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### Efficacy/Certification

- **bactericidal** incl. MRSA and suitable for hospitalism prophylaxis
- **fungicidal** (A. niger, C. albicans)
- **virucidal** according to EN 14476 all enveloped and non-enveloped viruses incl. polio
- **sporicidal** against C. difficile (EN 13704)

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**Manual cleaning and disinfection of instruments**

CE 0123 AlproZyme 

*Alkaline-enzymatic cleaning granulate for the pre-cleaning and pre-disinfection for the processing of medical and dental instruments, rotating precision instruments and endoscopes. Useable by soaking bath method as well as in ultrasonic devices and washer-disinfectors.*

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**Efficacy**

- removes powerfully and gently proteins, residues of tissues, biofilm etc. from surfaces of medical devices
- tested according to ISO/TC 198/WG 13 chap.6.2
- high staff protection (TRBA 250), **virucidal according to EN 16777** against non-enveloped adeno-, noro- and parvo-viruses as well as all enveloped „blood-borne“ viruses as HBV, HCV, HIV, Influenza at 1 % concentration before comprehensive disinfection
- optimal pre-treatment, in combination with BIB forte eco comprehensively bactericidal and virucidal

**Active ingredient basis**

Phosphates, tensides, parabens, defoaming agents, stabilizers, coated enzymes.

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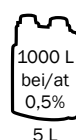
CE 0123 **BIB forte eco**

Liquid concentrate free of aldehydes, phenols and QAC for the non protein fixing cleaning and disinfection of medical and dental instruments, rotating precision instruments and endoscopes. Tested on instruments of company Komet, furthermore approved and recommended by Acurata, Busch & Co, Dentsply Maillefer, VDW, Helmut Zepf, Hager & Meisinger and Oridima.

### Efficacy

- **in immersion bath**
  - acc. to EN (bactericidal incl. TBC + yeasticidal) 0,5 % - 60 minutes  
2,0 % - 30 minutes
  - + limited spectrum of virucidal activity according to prEN 16777
- **in ultrasonic device**
  - acc. to EN (bactericidal incl. TBC + yeasticidal) 3,0 % - 10 minutes  
4,0 % - 5 minutes
- limited spectrum of virucidal activity according to EN 14476 (40 °C)
- virucidal according to EN 16777 („high level“ according to DVV 2012) after the pre-cleaning with AlproZyme

### Durability in use

7 days at normal contamination.

### Material compatibility

The extremely high corrosion-inhibiting properties could be improved by the QAC-free formulation, thus instruments sensitive to corrosion get optimal protection. Even with occasional exceeding of contact time BIB forte eco is absolutely compatible with the materials.

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**BIB forte eco**

**REF 3741** 1 L\*

**REF 3742** 5 L\*

**REF 4018** 4 L Wanne/4 l Tray\*

### Efficacy

- **bactericidal** (incl. TBC and suitable for hospitalism prophylaxis)
- **yeasticidal**
- limited spectrum of virucidal activity **according to EN 14476** against enveloped viruses e. g. HBV, HCV, Influenza as well non-enveloped viruses at 40 °C
- **virucidal according to EN 16777** after the pre-cleaning with AlproZyme incl. parvoviruses
- CJD-prophylaxis, meets the requirements of the RKI (pH ≥ 10)
- VAH/DGHM listed – instrument disinfection at high organic load

### Active ingredient basis

Modern combination of alkylamine and biguanide.

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- reinigend
- Bakteriostatisch
- Fungistatisch

### Efficacy

- cleaning
- bacteriostatic
- fungistatic

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*Non protein fixing solution ready for use for the pre-cleaning of medical instruments. Dissolves residues of blood and other body fluids from surfaces of medical devices. Approved amongst others by Planmeca.*

### Active ingredient basis

Complexing agent (EDTA), parabens, biguanides in aqueous solution.

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	IC-100
REF 4658	500 ml*
REF 4659	5 L*

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Biguanide, PHB-Ester.

*Solution ready for use for the non protein fixing inner cleaning prior to disinfection/sterilization of medical and dental hollow part instruments as turbines, hand pieces and contra-angles, internally cooled instruments and endoscopes.*

*WL-clean is a medical device of class I. The proof of suitability for cleaning has been given according to the series of standards EN ISO 15883 and according to the specifications of the German Society for Sterile Goods Supply (DGSV) – and has been confirmed by an expert's report.*

*Tested and approved amongst others by KaVo, NSK, SciCan, W+H.*

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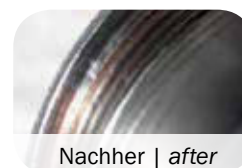
### Efficacy

- cleaning
- bacteriostatic
- fungistatic
- tested under practice conditions with soiling test according to EN ISO/TS 15883-5

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### Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virucidal**: according to EN 14476 and EN 16777 (SV 40) with non-enveloped polio-, adeno- and noro-viruses as well as all enveloped incl. blood-borne viruses, such as HBV, HCV, HIV and influenza.
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards

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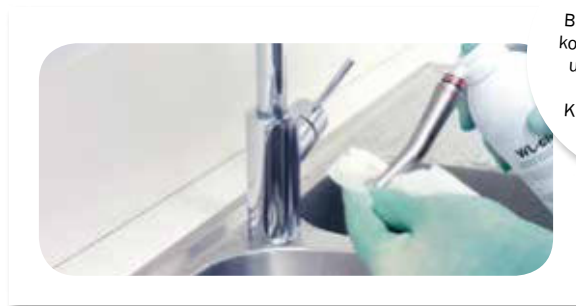
*Solution ready for use for the inner disinfection of medical and dental hollow part instruments as turbines, hand pieces and contra-angles after the non protein fixing pre-cleaning with WL-clean. WL-cid is a medical device of class IIb. The disinfection capacity under simulated practical conditions according to VAH/DGHM has been confirmed by an expert's report.*

### Active ingredient basis

*Aldehyde-free and phenol-free disinfection solution based on a combination of alcohols, QACs and alkylamines.*

**Important note:**  
For the correct reprocessing, please refer to our latest SOP

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- REF 4141** Kombipackung/Combination set WL-cid + WL-dry\*
- REF 4140** 4 x 500 ml WL-cid\*

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*Medically clean drying spray for quick drying and support of the cleaning of hollow part instruments such as turbines, contra-angles, hand pieces, internally cooled instruments, endoscopes and air scaler tips, after use of WL-clean/WL-cid.*

WL-dry contains: 1,3,3,3-Tetrafluorprop-1-en  
(R1234ze; CAS-nr.: 29118-24-9)

### Field of application

*Removing of residues of cleaning and disinfection agents and maintenance products and drying of spray channels and inner surfaces.*

### Content

*100 % chemical inert propellant, meets the hygienic requirements on „medical air“ for the processing of medical hollow part instruments.*

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*The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of hollow part instruments and transmission instruments after cleaning and disinfection with WL-clean/WL-cid<sup>1</sup>. WL-Blow is intended for use in the processing room and can be connected to the existing oil-free dry air compressor<sup>2</sup>.*

<sup>1</sup> Please note manufacturers` approvals!

<sup>2</sup> Connection of a bacterial filter is recommended!

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*The rinsing device „WaterJet“ with integrated water flow limiter is used to pre-rinse and facilitates the cleaning before the further processing of outer and inner surfaces of dental transmission instruments and general instruments. The rinsing device “WaterJet” is designed for the use in the preparation room and can be connected to the drinking water line.*

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### instrument disinfection

#### Virucidal final disinfection:

After pre-cleaning and general disinfection (e. g. with PlastiSept eco Wipes/MinutenWipes) soak a suitable cellulose wipe with undiluted BC-San 100, thoroughly wipe off the surfaces to be disinfected and let it take effect at least 5 minutes.

Disinfection effect tested with parvoviruses and *B. subtilis* spores.

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*Adapter for contra-angles and hand pieces with ISO-interface for complete internal cleaning/disinfection incl. head of instrument.*

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*Adapter for turbines with Sirona interface for complete internal cleaning/disinfection incl. head of turbine.*

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*Adapter for turbines with Multiflex-(KaVo/SciCan) interface for complete internal cleaning/disinfection incl. head of turbine.*

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*Adapter for turbines with Bien Air-interface for complete internal cleaning/disinfection incl. head of turbine.*

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*Adapter for turbines with Roto-Quick-interface (W&H) for complete internal cleaning incl. head of turbine.*

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*Adapter for longshank contra-angles T1 Classic (Sirona) for complete internal cleaning/disinfection incl. head of instrument.*

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*Adapter for turbines with NSK-interface for complete internal cleaning/disinfection incl. head of turbine.*

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*Adapter for turbines with Morita-interface for complete internal cleaning/disinfection incl. head of turbine.*

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*Adapter for ACTEON-AIR N GO Airpolisher Supra/Perio nozzle for complete internal cleaning/disinfection incl. head of instrument.*

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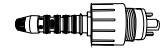
Set REF 3541

(Adapter 02/G, 04/G) inkl. 1 Set Ersatz-O-Ringe.


*(adapter 02/G, 04/G) incl. 1 replacement set of o-rings.*

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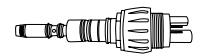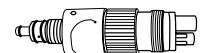
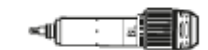
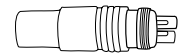
\* Please note manufacturer approvals!

## Manuelle Instrumentenreinigung und -desinfektion / Manual cleaning and disinfection of instruments

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*Rinsing adapter for ultrasonic tips/sonic tips M 3 internal thread from EMS, Mectron, KaVo, Komet.*

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*Rinsing adapter for ultrasonic tips M 3/a internal thread from Satelec, NSK.*

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*Rinsing adapter for ultrasonic tips in the Piezosurgery internal thread from NSK (VarioSurg), Mectron (Piezosurgery).*

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*Rinsing adapter for ultrasonic tips/sonic tips in the ultrasonic surgery external thread from Sirona (Siroprep and Siroimplant).*

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*Rinsing adapter for ultrasonic tips in the ultrasonic surgery, internal thread from Acteon, (2<sup>nd</sup> generation) e. g. for Piezotome.*

**Adapter U-1 + U-1a Kombi\* (Edelstahl)**  
**Adapter 1-U + U-1a combi\* (stainless steel)**

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*Combined adapter to clean and disinfect surgical contra-angles and handpieces externally cooled with the WL-series. Consisting of the U-1 rinsing adapter for ultrasonic tips and U-1a screw attachment for hose plug connection.*

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#### Universal adapter

WL-Adapter 01\* (plastics)

Universal adapter with Luer connector for transmission instruments (ISO 9168) for blowing-out of cleaning and maintenance products (WL-clean, WL-cid, maintenance oil) with WL-dry/WL-Blow. Also suitable for blowing out by using the multifunction syringe and to bring in the cleaning solution WL-clean via Luer connection e. g. into general hollow part instruments.

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Adapter only for spray water and spray air channels of all contra-angles and hand pieces with ISO-interface.

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Adapter only for spray water and spray air channels of turbines with Sirona-interface.

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Adapter only for spray water and spray air channels for turbines with Multiflex-(KaVo/SciCan) interface.

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Adapter only for spray water and spray air channels for turbines with Roto-Quick-interface (W&H).

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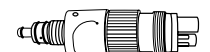
### Dosage

Dosage: 0.3 % - 0.5 %  
Ultrasonic device: 5 - 10 ml/l

Fill the concentrate into the door dosage system for the cleaner and close the flap. For automatic dosage systems connect the dosage lance with machine-compatible can and if necessary adjust the dosage quantity.

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*Mild alkaline liquid cleaning concentrate for the application in automatic and semi-automatic dosing systems of washer-disinfectors, for the automatic processing of medical devices.*

### Field of application

- alkaline cleaner (pH 10) for very effective removal of blood and protein residues at all temperature programs
- especially gentle to materials
- protects instruments made of steel and stainless steel from corrosion
- suitable in combination with TR-Neutralizer also in washer disinfectors as e. g. Miele thermodesinfectors. Approved by Melag.

### Active ingredient basis

Optimized cleaning concentrate based on mild alkaline tensides, QAC, complexing agents and phosphonates.

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*Liquid concentrate for the neutralization after the alkaline cleaning in washer-disinfectors.  
Approved amongst others for Miele Professional G 7735, G 7781, G 7881, SIROCLEAN, MELAtherm.*

#### Field of application

*Neutralization component based on phosphoric acid for use in washers and washer disinfectors during thermal preparation in the Vario TD program or RKI procedure for neutralization after the alkaline cleaning.  
**Also suitable for basic cleaning of dental instruments as immersion bath (see instructions of the manufacturer).***

#### Active ingredient basis

*Universally applicable neutralizing solution for all alkaline cleaning concentrates based on phosphoric acid.*

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*Coarse-grained special salt for water softeners in washers and washer disinfectors. Prevents calcium deposits in the equipment and on the wash items.*

#### Inhalt

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#### Content

Highly purified NaCl, without suspending agent

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*RegClean is designed for the intensive cleaning and regeneration of washers/washer disinfectors (chemical/chemical-thermal) and maintains the efficiency of the devices. RegClean removes powerfully bone substances, tissue and lime residues having formed in the spray arm and filter of the washer/washer disinfectant.*

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### Active ingredient basis

Highly effective cleaning granulate for the cyclic cleaning of the inside, lines and sump pit based on sulfamic acid.

- Anwendung von RegClean erfolgt ohne Beladung.
- The application of RegClean is carried out without load.

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### Efficacy

- **bactericidal**
- **yeasticidal**
- **virus-inactivating: HCV (BVDV)**
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for the disinfection of instruments

### Compatibility

AlproJet-D is highly compatible with materials such as plastics, rubber and non-ferrous metals, alloyed steels s.o. Approvals from leading device manufactures.

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*Disinfecting liquid concentrate for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator. Approved amongst others by Dentsply Sirona.*

### Application

*For the reliable cleaning, disinfection and care it is recommended that AlproJet-D is used on 3 days a week, 1 – 2 x per day (at noon and in the evening). For one application 2 L of a 5 % solution are needed. Let take effect for at least 30 min.; rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.*

### Active ingredient basis

*Highly effective aldehyde-free, phenol-free and chlorine-free cleaning- and disinfection concentrate based on QAC, tensides and alkaline carriers.*

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*Liquid concentrate with enhanced disinfecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator. Approved amongst others by Dentsply Sirona.*

### Application

*For the reliable cleaning, disinfection and care it is recommended that AlproJet-DD is used on 3 days a week, 1 – 2 x per day (at noon and in the evening). For one application 2 L of a 2 % solution are needed. Let take effect for at least 60 min. (DVG 2012); rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.*

### Active ingredient basis

*Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate with extended disinfection effect based on QAC, amines, tensides and alkaline carriers.*

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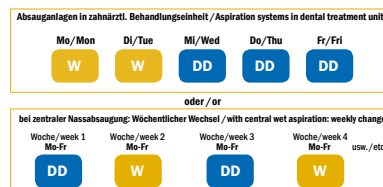
### Kalender/Calendar

### Efficacy/Certification

- **bactericidal**, EN practical test with protein load (EN 13727)
- **yeasticidal**, EN practical test with protein load (EN 13624)
- **virus inactivating** limited spectrum of virucidal activity according to EN 16777 against non-enveloped adeno- and noro-viruses as well as all enveloped „blood-borne“ viruses HBV, HCV, HIV, and influenza
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for the disinfection of instruments

### Compatibility

Very good compatibility with materials. Approvals from leading device manufactures.

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- AlproJet-DD**
- REF 3120** 1 L\*
  - REF 3115** 5 L Canister\*
  - REF 3116** 5 L Cubitainer\*
  - REF 3125-N** AlproJet-DD StarterSet\*
  - REF 3085** AlproJet MixCup\*

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### Efficacy

- against pseudomonas bacteria
- in combination with AlproJet-D/AlproJet-DD comprehensively cleaning and disinfecting

### Compatibility

Very good compatibility with materials.  
Approvals from leading device manufactures.

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*Liquid concentrate for the weekly acid contra-cleaning and maintenance of dental aspiration systems with and without amalgam separator.*

*Approved amongst others by Dentsply Sirona.*

### Application

*For the reliable cleaning and care it is recommended that AlproJet-W is used on 2 days a week, 1 – 2 x per day (at noon and in the evening) instead of AlproJet-D or AlproJet-DD. In case of high protein load AlproJet-W can also be used temporarily every day. For one application 2 L of a 5 % solution are needed. Let take effect for at least 30 minutes.; rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.*

### Active ingredient basis

*Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate for the acid contra-cleaning based on sulfamide acid, QAC and tensides.*

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*Liquid cleaning agent ready for use with disinfecting effect for the spittoon bowls of dental units. Approved amongst others by Dentsply Sirona.*

### Application

*Spread AlproCleaner generously (10 – 20 ml) in the spittoon bowl and remove visible residues with a plastic brush. Rinse with clear, cold water.*

### Active ingredient basis

*Quick-acting, alkaline cleaning solution for the direct application based on alcohol, QAC, tensides and complexing agents.*

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### Efficacy/Certification

- **bactericidal** incl. TBC
- **yeasticidal**
- **virus inactivating** active against all enveloped incl. „blood-borne“ viruses such as HBV, HCV, HIV, influenza, BVDV and vaccinia
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN standards for the disinfection of instruments

### Durability in use

Durability in use 2 days with normal contamination (scientific approval).

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*Solution ready for use, free of aldehydes and phenols, for the cleaning and disinfection of all kinds of dental impressions, worn dental prostheses and dental works.*

*PrintoSept-ID can also be used in disinfection devices for the processing of impressions, models, bite templates, auxiliaries, etc.*

### Contact times (undiluted)

- |   |        |
|---|--------|
| • bactericidal  | 1 min. |
| • yeasticidal   | 1 min. |
| • tuberculocidal  | 5 min. |
| • virus inactivating active against all enveloped viruses | 5 min. |

### Active ingredient basis

*Universally applicable, quick-acting, aqueous-based solution ready for use with amines, QAC, alcohol, tensides and complexing agents.*

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*Liquid concentrate for the cleaning of impression trays, mixing spatula and instruments.  
Aktiv solution can also be used in ultrasonic devices.*

### Active ingredient basis

*Highly effective, pH-neutral cleaning concentrate for the dissolving of organic and inorganic residues, based on non-ionic and anionic tensides and complexing agents.*

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### dental prostheses cleaner

Liquid concentrate for the cleaning and self-acting removal of acid-soluble deposits (e. g. tartar and cement residues) from medical devices such as dental prostheses and instruments.

#### Active ingredient basis

Acidic high-performance concentrate for the cleaning and passivation alloyed medical steels based on phosphoric acid and derivatives, tensides and polyols, ideal for cleaning of prostheses.

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*Solution ready for use for the cleaning of the sterilizing chamber, instrument trays and steel surfaces.*

#### Field of application

- For the removes of residues of blood, disinfectants (instrument disinfection) and tissue residues
- Removes powerfully in combination with SteriSponge all residues, which have formed during the sterilization process, even if they are old or heavily burned into material
- Cleans powerfully with best material compatibility (as it is free from grinding parts)

#### Active ingredient basis

*Quick-acting, alkaline solution for the support of the mechanical cleaning based on alcohol and tensides.*

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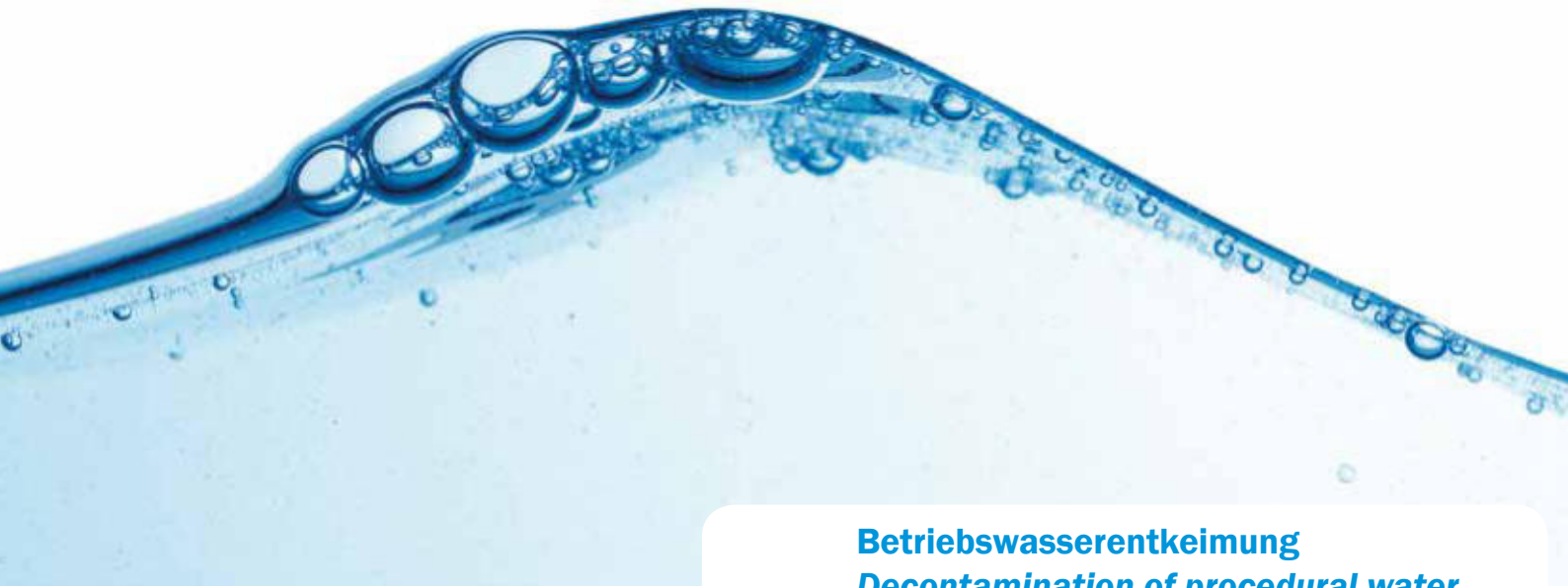
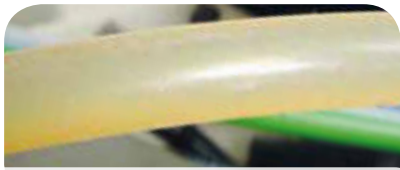
*Liquid concentrate for the continuous maintenance of the water quality in procedural water lines (including instruments and tumbler filler) of medical and dental units.*

*Approved by notable dental unit manufacturers, amongst others Dentsply Sirona.*

### Active ingredient basis

*Disinfection concentrate for the conditioning of the procedural water in medical units based on biguanides, complexing agents and tosylchloramide sodium.*

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*Solution ready for use for the decontamination and prevention of bio-film formation in procedural water lines of medical and dental units. Application of Bilpron at least once to twice a week in the Weekend-System II or ALPRO-BCS as well as other autarkic medical water supply systems removes effectively organic and inorganic residues in procedural water lines (including the instruments and the tumbler filler). The contact time for the disinfection solution ready for use should be at least 12 hours (overnight) in the procedural water lines, it can also be used as long term disinfection (holidays) up to 3 months.*

### Active ingredient basis

*Solution for the sanitation, preservation and cleaning of the procedural water lines of medical units based on propylene glycol, PHB-esters, biguanides and complexing agents in an aqueous solution.*

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	<b>Bilpron</b>
<b>REF 3181</b>	6 x 1 L*
<b>REF 3081</b>	500 ml Messbecher/Cup*
<b>REF 3179-N</b>	Bilpron StarterSet*
<b>REF 3179-PM</b>	Bilpron StarterSet 3 (PM)*

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### Efficacy

- bactericidal
- fungicidal

Approved for idle time/long term disinfection amongst others by Chirana-Dental, Heka Dental, Planmeca, Ritter, Dentsply Sirona, Takara Belmont, TGA, EMS, NSK, Mectron, Ultradent.

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*Dip slide for semi-quantitative monitoring of microbial load of water and surfaces.*

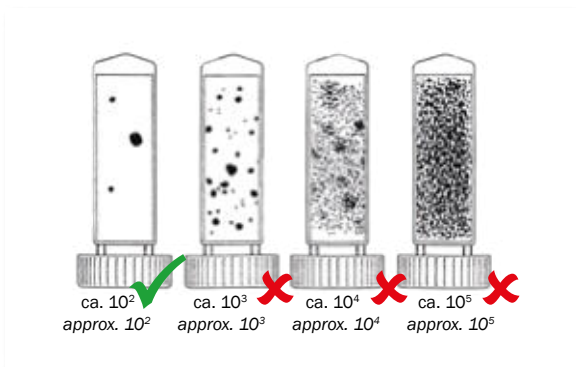
#### Field of application

*The dip slide is used for the semi quantitative monitoring of the microbial load of liquids and surfaces in industry, environment and medicine.*

*The microbial count in liquids is measured in "colony-forming units" (CFU) per ml, on surfaces in CFU per cm<sup>2</sup>.*

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Liquid preparation for the virucidal final disinfection of surfaces of medical devices, such as light guides, polymerisation lamps, heads of intraoral cameras/scanners, as well as for the disinfection/processing of wipe dispenser boxes and pressure cylinders (1 - 1.5 L) of stand-alone procedural water supply systems such as the ALPRO-BCS (BottleCareSystem).

### Active ingredient basis

Concentrate for the routine processing of ALPRO-BCS (BottleCareSystem) based on NaOCl.

### Application

**Pressure cylinders of procedural water supply systems:**

Pour the content of the bottle (100 ml) of **BC-San 100** into the empty pressure bottle and fill it with drinking water up to the thread. Leave the solution in the pressure bottle to for at least 6 hours (e. g. overnight). During disinfection, keep the filled pressure bottle closed in the processing room or mount it as usual to the holder of the procedural water supply system.

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*Procedural water supply system for medical and dental treatment units. Meets the requirements according to **DIN 1988-100/EN 1717** and provides an easy hygiene management of the dental unit water lines and the connected dental unit.*

#### Field of application

- Meets the standards DIN 1988-100 and EN 1717
- Each dental unit can be individually upgraded
- For the treatment processed tap water (with Alpron) or drinking water can be used
- The disinfection of the water lines with Bilpron is carried out weekly in idle times (at least for 12 hours) or during vacations

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*Chair-side device for feeding the procedural water lines of medical and dental units with Bilpron.*

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### Field of application

- Easy handling
- High degree of safety and longevity
- Can be connected to every dental unit without an own decontamination system
- The disinfection of the water lines with Bilpron is carried out weekly or during vacations

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*Multifunction container for  
the cleaning and maintenance  
of the aspiration system*

*For 3, 5 and 10 l canisters*

*For 5 and 10 l cubitainers*

*Helps to unscrew the cap of  
a canister for 3, 5 and 10 l*

*Helps to unscrew the cap of  
a cubitainer*

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*For Bilpron/Weekend-System II  
and dosage of other disinfection  
solutions*

*Plastic, vol. 1 l  
For the reprocessing  
processing up to 60 °C in RDG*

*Plastic, vol. 4 l  
For the reprocessing  
processing up to 60 °C in RDG*

*External housing for retrofitting of  
ALPRO-BCS*

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*For 1 l Alprox  
round bottle*

*For 500 ml spray  
bottle  
(REF 3003)*

*500 ml empty spray  
bottle with spray head (for  
MinutenSpray-classic)*

*Special sponge for  
cleaning autoclaves with  
SteriCleaner*

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*wall holder (round  
dispenser boxes) for  
JumboWipes 20  
and 50*

*wall holder (round  
dispenser boxes) for  
MaxiWipes reel/  
MaxiWipes-L reel*

*wall holder (square  
dispenser boxes) for  
MaxiWipes*

*Multipurpose dispenser  
for soaps and alcoholic  
solutions, drip tray  
inclusive. Suitable for  
1 l and 500 ml Euro  
bottles*

*„Touchless mini“  
Automatic Dispenser  
System*

*Replacement pump for  
„Touchless mini“  
Automatic Dispenser  
System*

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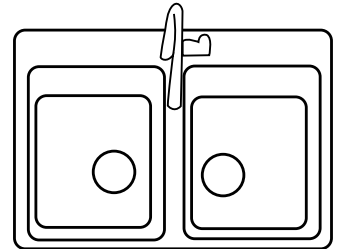
BIB forte  
Tauchwanne/

UNREIN dirty

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sterilization area  
authorized  
personnel only

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© Melag