L Protect

Original

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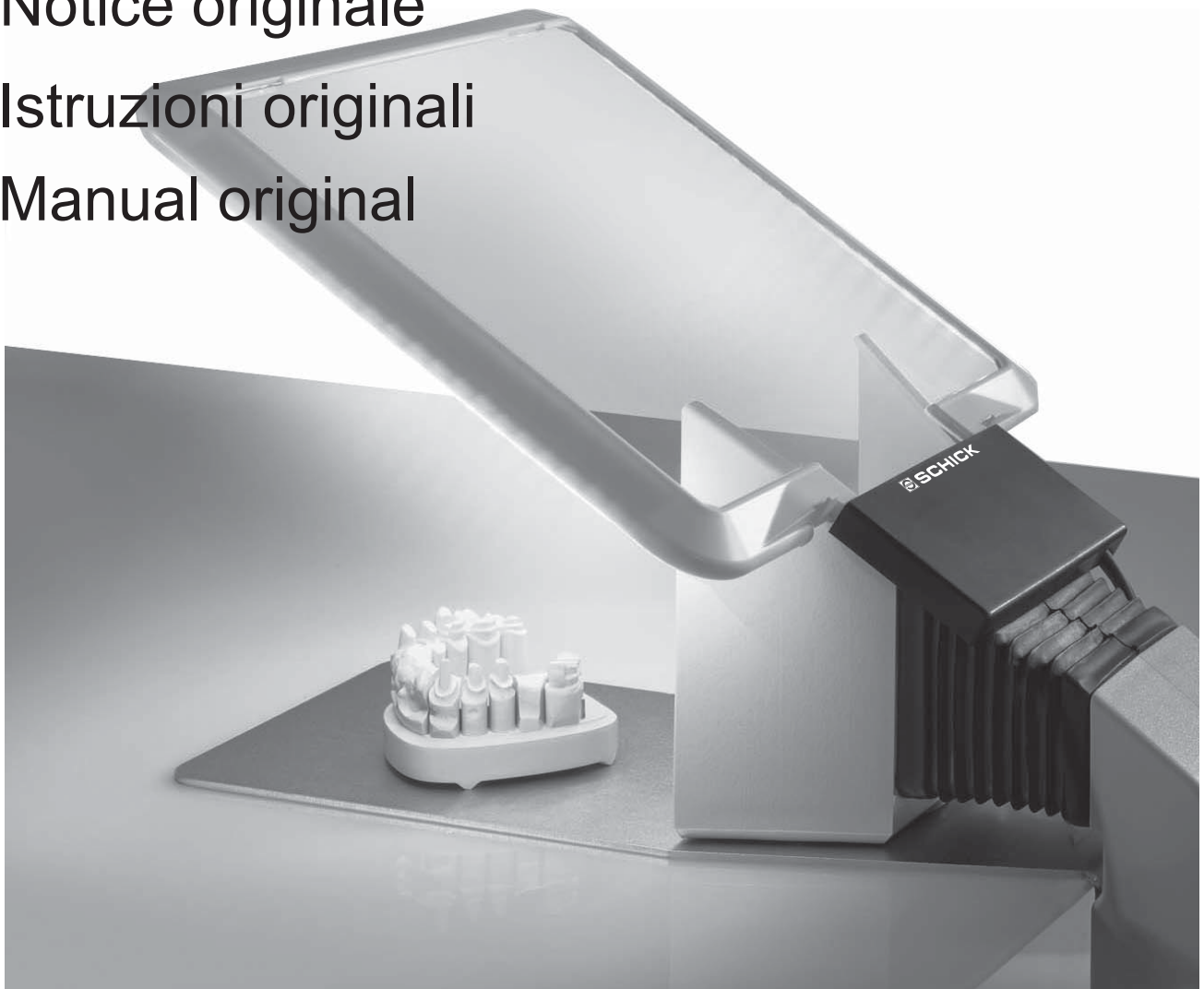
Instructions

Notice originale

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Manual original

D

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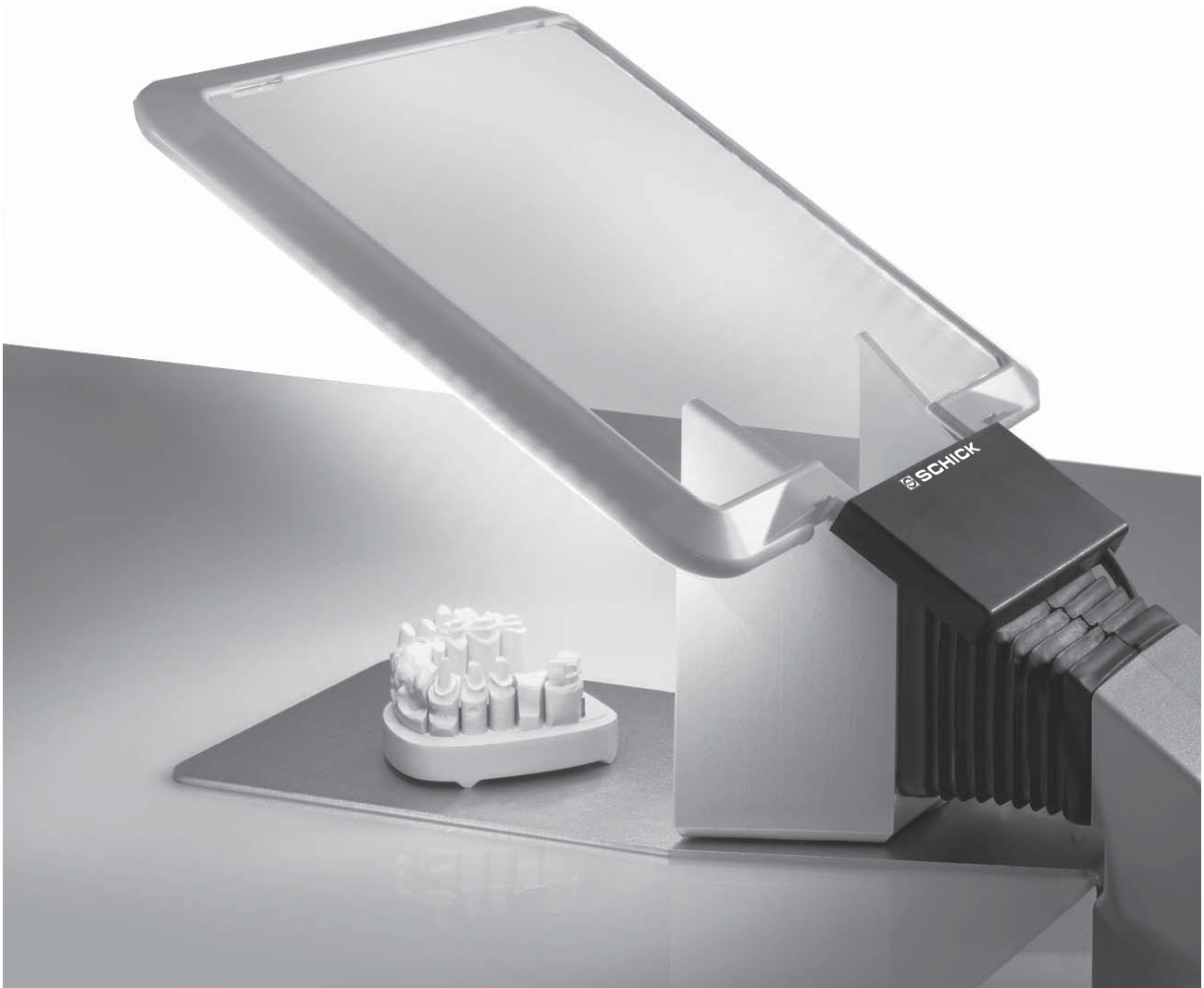
Instructions

E

We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new **L Protect** .

We wrote these instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

Index page

1. Safety instructions.....	3
2. Range of applications.....	3
3. Summary - List of contents.....	3
4. Accessories / Optionals	4
5. Initial start up and installation.....	4
5.1 Installation without adapter.....	4
5.2 Installation with adapter.....	5
5.3 Installation on the bracket.....	5
5.4 Switching on and off	5
6. Exchange of the protection glass.....	6
7. Errors.....	6
8. Technical data.....	7
9. Overview suction funnels.....	7
10. Declaration of conformity.....	8

Maximum illumination and protection

L Protect by Schick Dental – the worldwide first protective glass with LED lighting. It offers a unique combination of modern, energy-saving workstation illumination and a protective glass pane. Equipped with highly efficient SMD-LED's, **L Protect** offers exceptional illumination of the working area without irritating shadows. At the same time it ensures a maximum occupational safety.

Universal application

The high quality protective glass with LED frame can be fitted to nearly all conventional suction funnels with a retaining adapter. It can also be used without a suction funnel with the optionally available bracket. The LED frame and protective glass can be easily separated. With an illuminance of 2,500 Lux (brighter than two 60W light bulbs) and minimum energy requirements of 8 watts, the **L Protect** is an economical alternative to conventional workplace lighting systems.

1. Safety instructions

1.1 Ascertain that your mains supply coincides with the data on the rating plate.

1.2 **L Protect** - units are not suitable for the following applications:

- in areas where there is a risk of explosion
- for medical applications
- for working on moist materials

Caution!

- When using rotary instruments, do not exceed the maximum speeds laid down by their manufacturer.
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the **L Protect** unit should it not have been used in accordance with the operating instructions.

2. Range of applications

L Protect is designed for universal use on the workbench of dental laboratories as additional light in combination with a protection glass pane.

L Protect can be fixed on most common suction funnels and ensures optimal shadowfree light combined with a face protection by the protection glass pane in accordance to DIN EN 12150. Due to the optional special bracket Art. No.: 4474 **L Protect** can be used without a suitable suction funnel.

Conditions of environment:

- interior 5° - 40° C.
- up to 2,000 meter over sea level

Categorie of overvoltage: II

Grade of pollution: 2

WEEE-Reg.-Nr. DE 78620387

3. Summary - List of contents

(1)

(2)

(3)

(4)

(5)

(7)

(8)

- | | | |
|-----|---|----------------|
| (1) | illuminated frame cpl.
(incl. protection glass pane) | art. No.: 4470 |
| (2) | protection glass pane | art. No.: 4468 |
| (3) | clip | art. No.: 4465 |
| (4) | panel-bracket | |
| | - left | art. No.: 4466 |
| | - right | art. No.: 4467 |
| (5) | AC-Adapter | art. No.: 4475 |
| (6) | cable holder 5x
(without pic.) | art. No.: 1574 |
| (7) | adapter | art. No.: 4469 |
| (8) | screws for Adapter | art. No.: 4508 |

4. Accessories / Optionals


bracket complete Art. No.: 4474
For using L Protect without suitable suction funnel


5. Initial start up and installation

5.1 Installation of the clip on the suction funnel without adapter

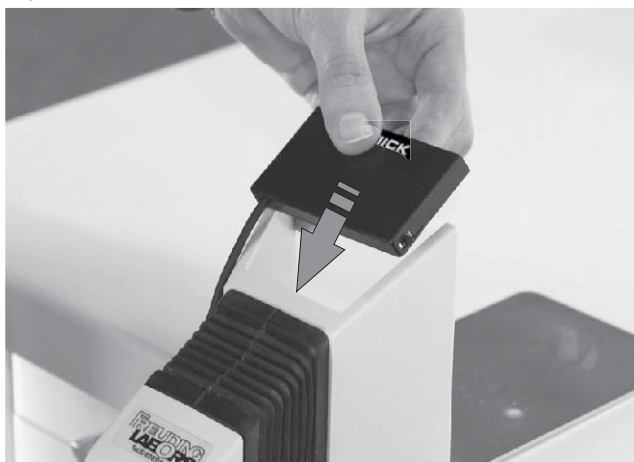Pic.1

Pic.2

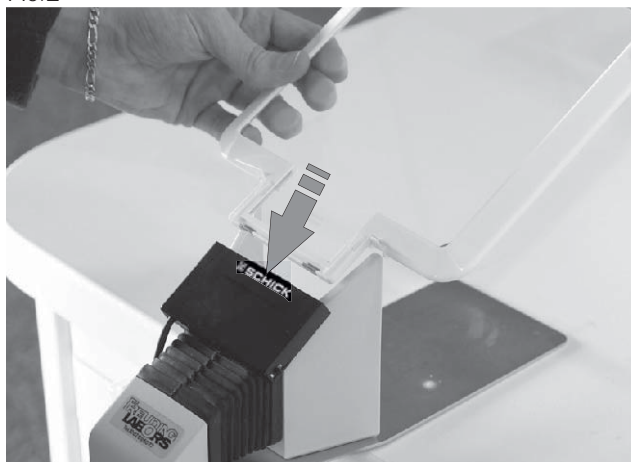The clip can be fixed on most common suction funnels without any additional adapter. When fixing the clip (pic.1) make sure the clip fits tight and safe. Afterwards the illuminated frame can be inserted into the clip (pic.2).

Pic.3

Pic.4

The power cable can be fixed to the suction funnel by the supplied cable holders. Fixing the cable as shown on pic. 3 + 4 allows an easy removal of the suction funnel. It is highly recommended to fix the power cable underneath the workbench!

5.2 Installation of the clamp (3) with adapter

On some suction funnels (as seen under point 10.) the clamp must be installed by using the supplied adapter. On some Zubler or Freudling suction funnels, the original screws for fixing the pane have to be replaced by the supplied flat head screws to make space for the clamp on the adapter (pic.5).

pic. 5

5.3 Using of **L Protect** with the optional bracket

L Protect can be used with the optional bracket Art.No.: 4474 without any suitable suction funnel. Mount the clip to the bracket as shown on the picture below.

pic. 6

5.4 Switching on and off

The lighting can be switched on and off easily by the switch on the right side of the clip.

pic. 7

6. Exchange of the protection glass pane

Excessive work on alloys can cause irreversible wear on the protection glass. If necessary the pane can be exchanged easily. For removing the pane from the illuminated frame, remove the panel-brackets first (pic. 5 + 6).

Pic. 5


Pic. 6

For the installation of the new pane slide the glass into the guidance grooves at the upper frame side and lay the pane into the frame completely (pic. 7).

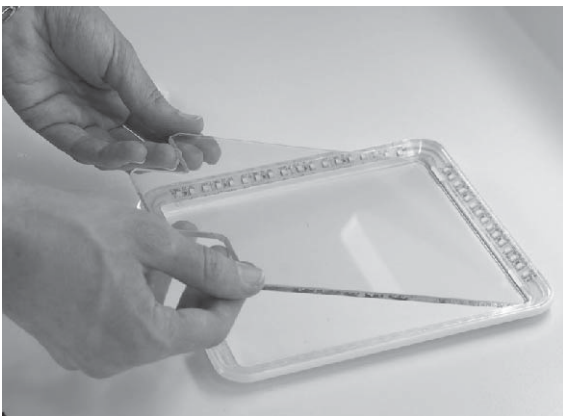Pic. 7

Pic. 8

Fix the pane with the supplied panel brackets as shown on the picture 8.

 Always use the original protection glass pane from Schick! The pane is made from safety glass and minimizes the risk of injuries in case of breakage.

7. Errors

Error description	Cause	Solution
flickering light	bad contact between illuminated frame and clip	<ul style="list-style-type: none"> - push illuminated frame completely into the clip - clean contacts from illuminated frame - clean inside of the clip
no light	bad power supply	<ul style="list-style-type: none"> - check connection from mains supply - push illuminated frame completely into the clip - check on/off switch position

8. Technical data

Dimensions illuminated frame with clip:

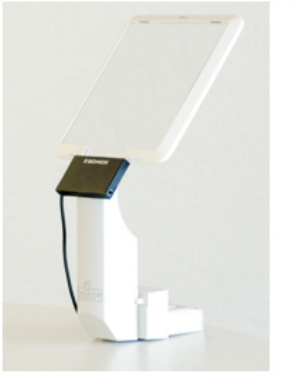
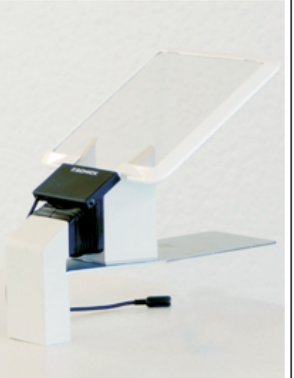
Height : 240 mm
 Widht: 190 mm

Illumination: 2500 Lux
 Colour temp.: 6000 K
 Light colour: daylight white

Dimensions protection glass pane:
 Height: 205 mm
 Widht: 160 mm
 Strenght: 4 mm
 Safety glass in accordance to DIN-12150-1

Voltage: AC 100 - 240 Volt
 Frequenzy: 50/60 Hz
 Intake power: 8 Watt
 Operatin voltage: 12 Volt

9. Overview suction funnels

Manufacturer:	KaVo	Freuding	Freuding/Zubler	Freuding
Type:	EWL 382	Variteam 1	A 90 / A 90 M(table top)	A 86
Picture:				
compatible:	Direct	Direct	Adapter necessary	Direct
Manufacturer:	KaVo	Strohm&Maier	Zubler	Renfert
Type:	Masterspace		R1200	Absaugmaul
Picture:				
compatible:	Direct	Adapter necessary	Adapter necessary	Direct

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declare herewith, that the product

L Protect consisting of

illuminated frame 4470
protection glass pane 4468
clip 4465
AC- adapter 4475

is in conformity with the following provisions of Directive:

2006/42/EG (machinery directive)

2014/30/EU (EMV directive)

2011/65/EU (RoHS)

Name and address of
person in charge:

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Schemmerhofen, March 2017

A handwritten signature in cursive script, appearing to read 'W. Schick'.

W. Schick
General manager

Subject to technical modifications without prior notice

This unit complies with the current VDE
(German association of electrical technicians)
regulations concerning safety and suppression.
These instructions should be readily
accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise
you that a proper repair service and
suitable qualified personnel are required for such
highly developed technical equipment.
SCHICK guarantees to carry out perfect
repairs using original spare parts

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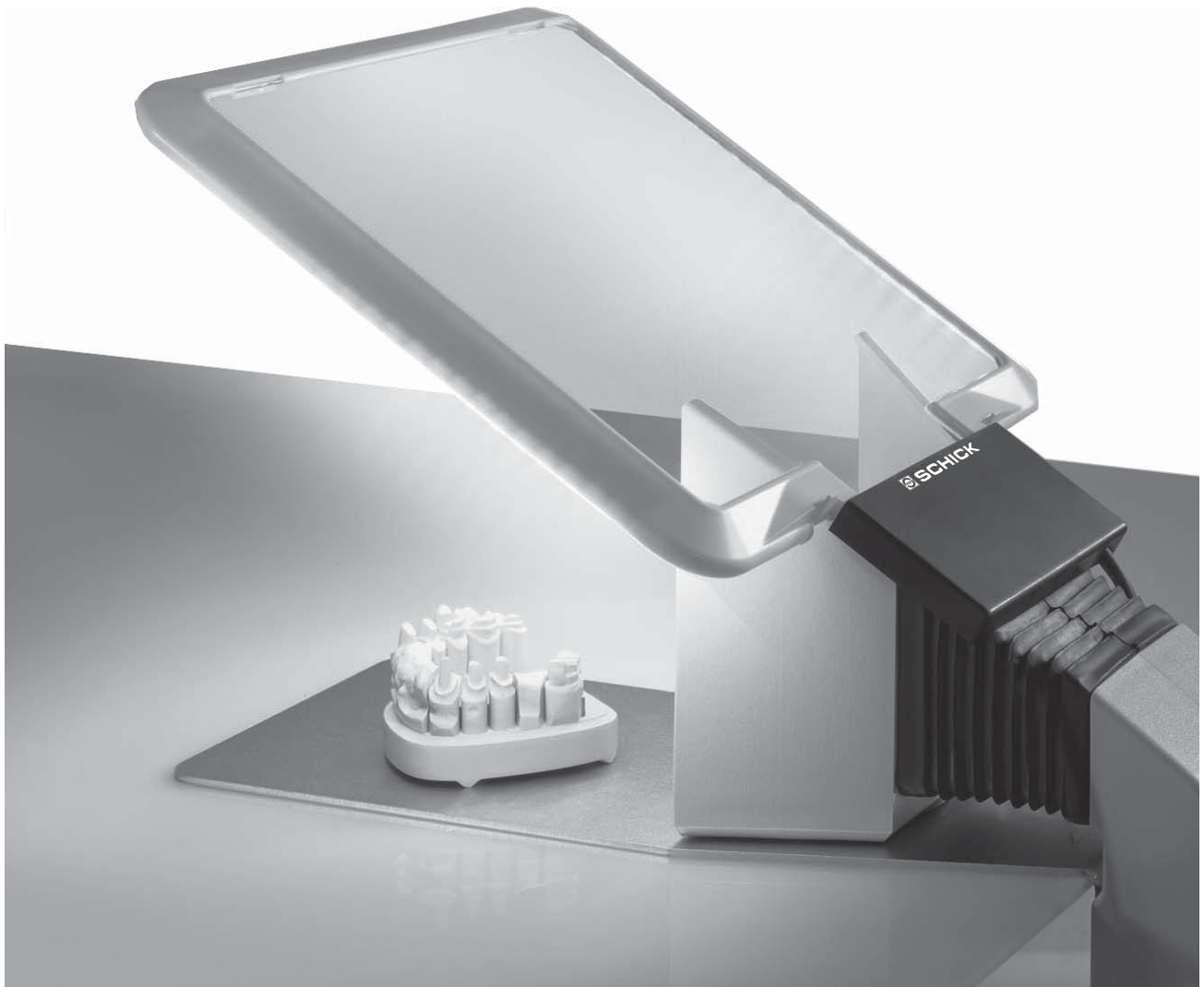
Schemmerhofen, March 2017

A handwritten signature in cursive script, appearing to read 'Schick'.

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This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression. These instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment. SCHICK guarantees to carry out perfect repairs using original spare parts

Manufacturer:

Schick GmbH
Lehenkreuzweg 12
D-88433 Schemmerhofen
Telefon +49 7356 9500-0
Telefax +49 7356 9500-95
E-Mail info@schick-dental.de
Internet www.schick-dental.de