Deutsch

Original

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## Elektrisches Wachsmesser

D

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A handwritten signature in cursive script that reads 'Schick'.

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Electric wax knife

E

We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new electric wax knife.

We wrote these instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

Index	page
1. Safety information.....	2
Caution!	
2. Range of applications.....	2
3. List of contents - accessories/spare parts..	3
4. Connecting the electric wax knife to the <b>QUBE</b> control unit.....	4
5. Activation and setting of the electric wax knife.....	4
6. Recommendations for adjustment of wax knife .....	5
7. Changing the modelling tip .....	5-6
8. Technical Data.....	6
9. Declaration of Conformity.....	7

## 1. Safety information

- 1.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 1.2 The electric wax knives are not suitable for the following applications:
  - in areas where there is a risk of explosion
  - for medical applications
  - for working on moist materials
- 1.3 Ensure that all regulatory requirements are observed during use
  - always wear protective glasses
  - provide enough light at the working place
  - use dust suction
- 1.4 When not in use, the wax knife should always be stored in the unit supplied so as to prevent possible contact with hot components.

### Attention:

- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the electric wax knife should it not have been used in accordance with the instructions.

## 2. Range of applications

The electric wax knife is designed for universal use in dental laboratories for the processing of dental waxes. The SCHICK wax knife can withstand temperatures of up to 200°C and can thus be used with all commercially available dental waxes.

The electric wax knife is operated via the **QUBE** workplace system and its control satellite with touch-sensitive keys.

Conditions of environment:

- interior 5° - 40° C.
- up to 2,000 meter over sea level

Categorie of overvoltage: II

Grade of pollution: 2

WEEE-Reg.-Nr. DE 78620387

### 3. Summary electric wax knife art.no. 9066 and accessories

List of contents:

electric wax knife art.no.: 9065/1

modelling tip 0,6mm art.no.: 9200/2

modelling tip 1mm art.no.: 9200/3

modelling blade art.no.: 9200/4


support wax knife art.no.: 9090/4

optional accessories:

wax tray art.no.: 9092

#### 4. Connecting the electrical wax knife to the **QUBE** control unit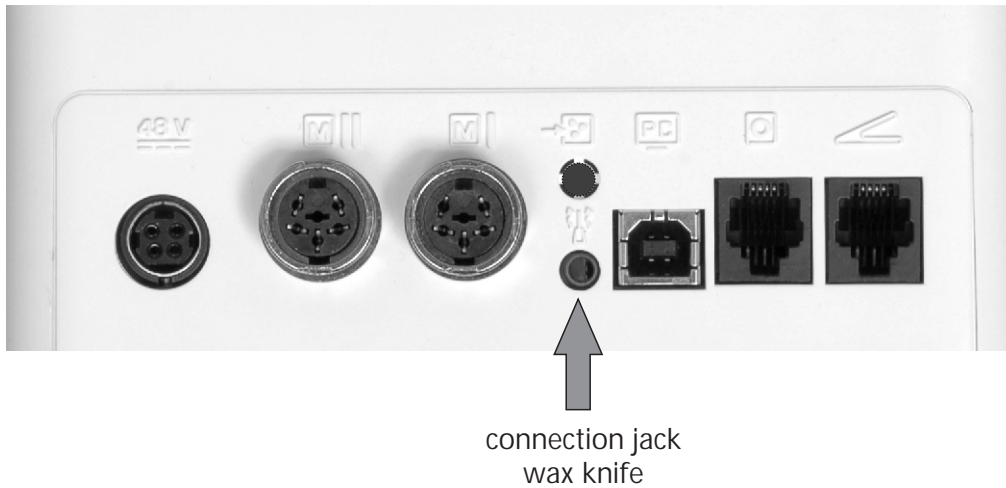


On the rear of the **QUBE** control device, there is a connection jack for the electric knife's power supply. Insert the wax knife's plug here.

#### 5. Activation and setting of the electric wax knife via the controls satellites on the QUBE workplace system

To activate/deactivate the wax knife on your **QUBE** controls satellite, take the following steps:

##### 5.1 Switching on and adjusting the wax knife (please see recommendations in point 6)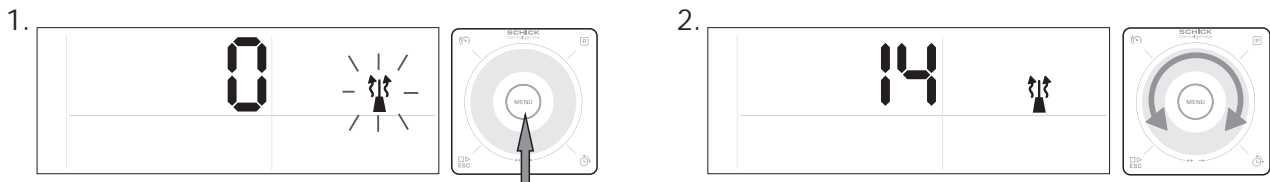

1. **Menu** Press menu for 2 seconds to switch to programming level

2. To activate the wax knife, rotate the wheel, set a value higher than 0 and confirm with **ESC**. The heat can be adjusted within a range of 1 - 20. This equates to a temperature range of between c. 60° and 200° C. To reset the value to 0, rotate and confirm with **ESC**.

##### Re-start from the memory:

If the QUBE was turned off without having the electric wax knife de-activated, the number that was used last stays in the memory.

To re-activate the electric wax-knife, enter the programming level as described under point 5.1/1. The number that was used last is shown by the flashing display. After leaving the programming level by pressing **ESC**, the electric wax knife is active.

## 6. Recommendations for adjustment of wax knife

The heat of the wax knife can be set to a range of values between 1 and 20 using the controls satellite, with 20 being the highest and 1 the lowest.


The following list of setting recommendations are examples which relate to the Schick DesignWachs and which can be used as a guide for settings to suit individual needs.

Wax types	Setting recommendation	Processing temperature
Cervical wax purple	range 10	approx. 125°C
Ceramic wax beige	range 11-12	aprox. 130° - 135°C
Modelling wax grey	range 15	approx. 140°C
Milling wax green	range 15-17	approx.. 140° - 150°C

## 7. Changing the modelling tip

### 7.1 Changing the modelling tip with support wax knife

1.

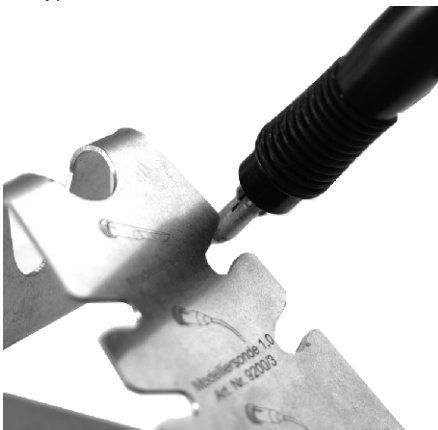2.

3.


Place the modelling tip sideways on one of the forked holding devices and pull the wax knife upwards. To fit a modelling tip, fix the wax knife directly from above onto the modelling tip and remove sideways from the fork.

Never place the hot wax knife on a flammable surface. Risk of fire!

## 7.2 Changing the modelling tip without support wax knife

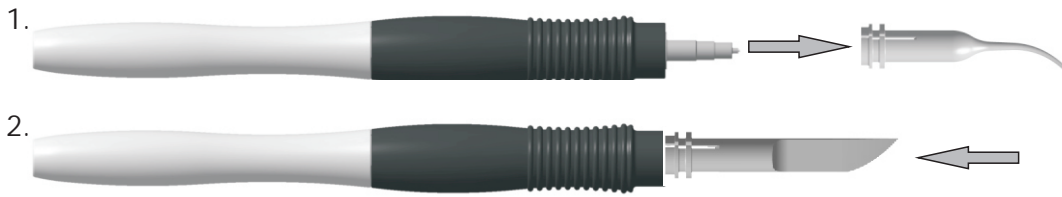1. Using a suitable gripper, pull the modelling tip in the direction of the arrow
2. Attach the modelling tip to the wax knife in the direction of the arrow, ensuring it is firmly connected.

Attention: The modelling tips attain temperatures of up to 200°C! Danger of burns!

## 8. Technical Data

### Electric wax knife

diameter:	14 mm
length without cable:	171 mm
length cable:	1,95 m
weight with cable:	67 g
Min./Max. temperature:	60°C /200°C
Changing modelling tip:	Quick change system

Subject to technical modification without prior notice

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declare herewith, that the product

Electric wax knife 9066 consisting of  
wax knife 9065/1 in connection with  
modelling tip 9200/2, 9200/3, 9200/4 and  
support wax knife 9090/4

is in conformity with the following provisions of Directive:

2006/42/EG	(machinery directive)
2014/30/EU	(EMV directive)
2011/65/EU	(RoHS)

Name and address of person in charge:	Wolfgang Schick Lehenkreuzweg 12 88433 Schemmerhofen
--	--

Schemmerhofen, March 2017

A handwritten signature in black ink, appearing to read 'Schick'.

W. Schick  
director

---

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These instructions should be readily accessible and are best kept close to the unit itself. We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts.

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Schick GmbH