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**GB****Instructions for use****USA**

**Remin Pro** is a water-based cream that contains hydroxyapatite, sodium fluoride (1,450 ppm fluoride) and xylitol. **Remin Pro** is available in melon, strawberry and mint flavour.

**Remin Pro** provides extra protection for teeth and, in doing so, it helps neutralise acids from acid-forming bacteria in plaque. **Remin Pro** additionally aids the neutralisation of other acids in the mouth.

**Indications:**

- After tooth whitening
- After professional tooth cleaning
- For the prevention and control of hypersensitivities
- During orthodontic treatment

**Application:**

1. Apply a bean-sized amount of **Remin Pro** to the teeth with a finger or suitable instrument (toothbrush or swab) and distribute.
2. The patient should distribute the remaining **Remin Pro** cream in his or her mouth with the tongue. **Remin Pro** and the saliva should remain in the mouth for as long as possible (minimum of 3 minutes) for an optimal result.
3. The patient should spit out the remaining amount. Rinsing should be avoided if possible. The patient should wait at least 30 minutes after the treatment before eating or drinking.

**Also for daily use after oral hygiene.****Storage:**

Store **Remin Pro** in a cool, dry area protected from direct sunlight (4 °C - 23 °C / 39 °F - 73 °F). Do not use the material after the expiry date.

**Precautions:**

1. **Remin Pro** contains parabens. **Remin Pro** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
2. Always close the tube immediately after use.
3. The application of **Remin Pro** is not recommended for patients under 6 years or for home use on children under 12 years.
4. In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention.
5. Immediately discontinue treatment and seek medical attention if symptoms of angioedema occur that could be attributed to the ingredients in **Remin Pro**.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

**CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.**

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

**Keep this material out of reach of children.**

**For dental use only.**

An explanation of the symbols used in labeling can be found at [www.voco.com/us/symbols](http://www.voco.com/us/symbols)

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Our preparations have been developed for use in dentistry.  
As far as the application of the products delivered by us is  
concerned, our verbal and/or written information has been  
given to the best of our knowledge and without obliga-  
tion. Our information and/or advice do not relieve you from  
examining the materials delivered by us as to their suitability  
for the intended purposes of application. As the application  
of our preparations is beyond our control, the user is fully  
responsible for the application. Of course, we guarantee the  
quality of our preparations in accordance with the existing  
standards and corresponding to the conditions as stipulated  
in our general terms of sale and delivery.

Last revised: 12/2016

VOCO GmbH Phone +49 (4721) 719-0  
Anton-Flettner-Str.-1/3 Fax +49 (4721) 719-140  
27472 Cuxhaven e-mail: marketing@voco.com  
Germany www.voco.com

VC 60 00 2006 E1 1216 99 © by VOCO