VOCO

GB

Traypurol tabs

Instructions for use

CE

Active-tabs for cleaning trays and instruments.

Traypurol tabs remove residue from alginates, zinc oxide eugenol, carboxylate and phosphate cements from impression trays and instruments without using ultrasound. Traypurol tabs are bio-degradable.

Method of use:

2 **Traypurol tabs** are needed for 1 litre of cleaning solution. Simply remove the tablets from the foil, place them in the cleaning container and fill with cold or warm water. Warm water accelerates the cleaning process. Avoid contact with skin.

Place trays and instruments in the tablet solution after removing any large pieces of material. Standing the trays upright promotes the cleaning process. Leave impression trays and instruments in the solution for at least 2 hours, but not more than 72 hours. Remove and rinse with clean water.

Traypurol tabs are suitable for all rustproof metals and plastics.

Change the cleaning solution every 2 - 3 days.

Traypurol tabs can be used in ultrasonic baths, but not in dishwashers.

Contains complexing agent > 30%, non-ionic tenside < 5%, corrosion inhibitors.

Store **Traypurol tabs** in a dry area at temperatures of 4°C to 23°C.

Warning:

Irritates the eyes.

Safety Advice:

In case of contact with the eyes, rinse immediately with plenty of water and consult a physician. If swallowed, seek immediate medical attention and present package.

Our preparations are developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 002356 E1 0711 99 © by VOCO